

Hand Control HB200 Wireless Data Sheet



Contents

Introduction.....	3
Features and options	3
Usage	3
Ordering example	4
System overview.....	5
Dimension drawing	6
Functionality.....	7
Locking	7
Magnet key	7
Battery	8
Pairing Bluetooth devices.....	9
Direct pairing.....	9
Front covers.....	10
Contacts	13

Introduction



The HB200 Wireless is a Bluetooth Low Energy (BLE) hand control for the medical and beds segments. It is available with up to 5 rows and locking of individual channels by using a magnet key. One LED will function as pairing and battery indicator.

Features and options

Housing colour	Light grey, RAL 7035
Protection class	IPX6
Number of buttons	Up to 10 buttons
Button type	Domes
Flammability rating	UL94 V-0
Weight	98 g
Front cover	Standard or customised
Attachment	Hook
LEDs	1
Battery life	Approx. 2 years at 140 sec/day

Usage

Operation temperature	+5 °C to + 40 °C
Storage temperature	-10 °C to + 50 °C
Relative humidity	20% to 80% non-condensing
Atmospheric pressure	700 to 1060 hPa
Approvals	- IEC 60601-1 - IEC 60601-1-6 - IEC 60601-1-2 - ANSI/AAMI ES60601-1 - CSA CAN/CSA-C22.2 NO. 60601-1 - RED 2014/53/EU - FCC Part 15.249 - IC RSS247 - Telec MIC


Ordering example

	HB	
Series	200	200 = HB200
Rows	3	1 = 1 Row 2 = 2 Rows 3 = 3 Rows 4 = 4 Rows 5 = 5 Rows
Functionality	V0	V0 = Bluetooth funct. V V3 = Bluetooth funct. V3 V5 = Bluetooth funct. V5 VX = Special
Bluetooth generation	4	0 = LINAK GEN1 4 = LINAK GEN2
Hall	1	1 = Hall
Not used	0	0 = None
Not used	0	0 = None
Not used	0	0 = None
Not used	0	0 = None
Front cover layout	3B	xB = Standard
Front cover number	001	XXX = Last 3 digits in front cover number
IP-Degree	6	6 = IPX6
Colour	2	2 = Grey (RAL7035)
Mounting	1	1 = Hook
Connectivity	2	2 = Bluetooth
Plug Type	0	0 = None
Cable	N	N = No cable
Cable colour	0	0 = No cable
Not used	0	0 = None
Not used	0	0 = None
Not used	0	0 = None
Not used	0	0 = None


Chosen item number: HB2003V04100003B00162120N00000

* For further BLE information, see sales backup 'Chapter 11.02 Bluetooth Low Energy'

System overview

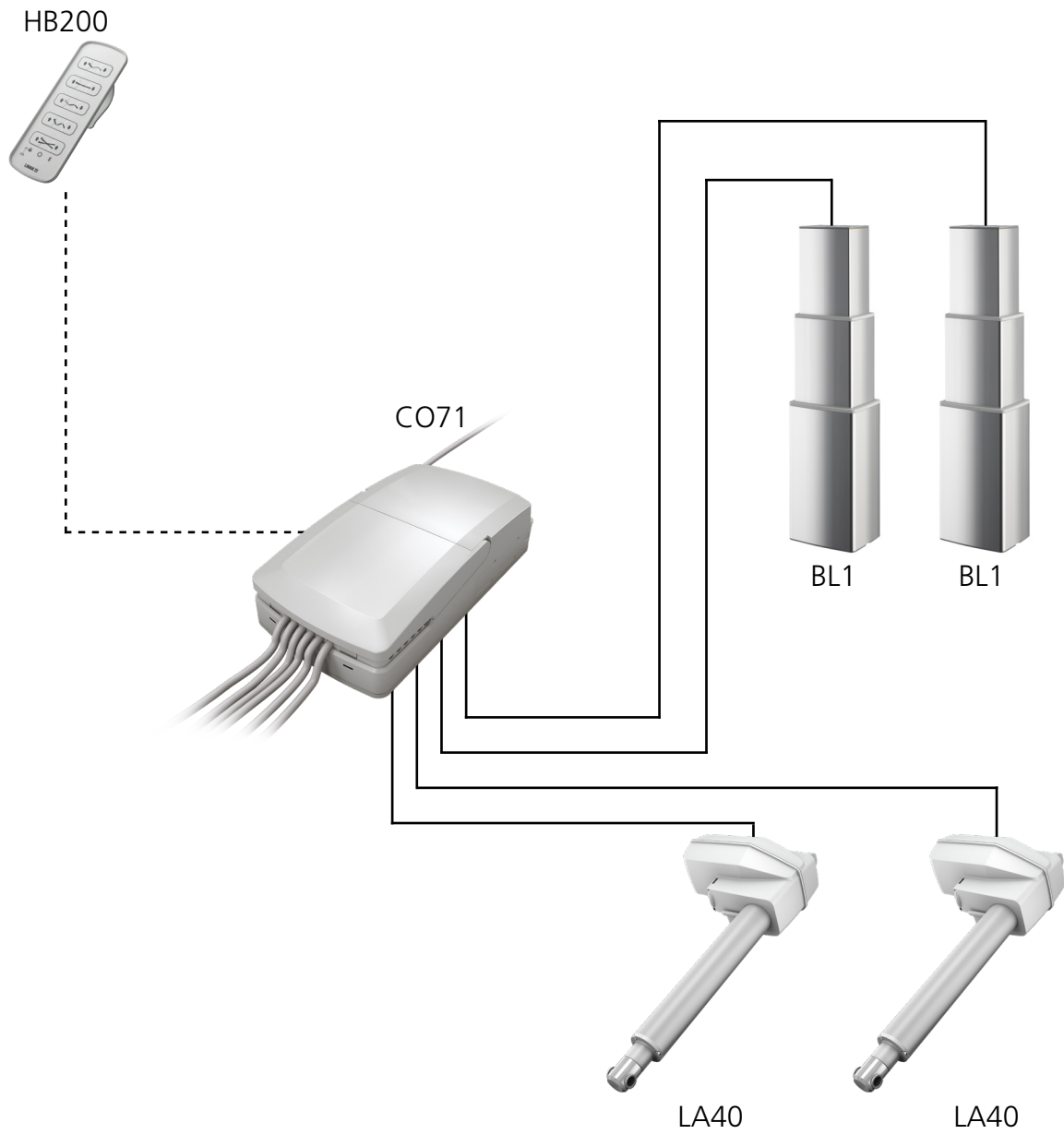

Dimension drawing

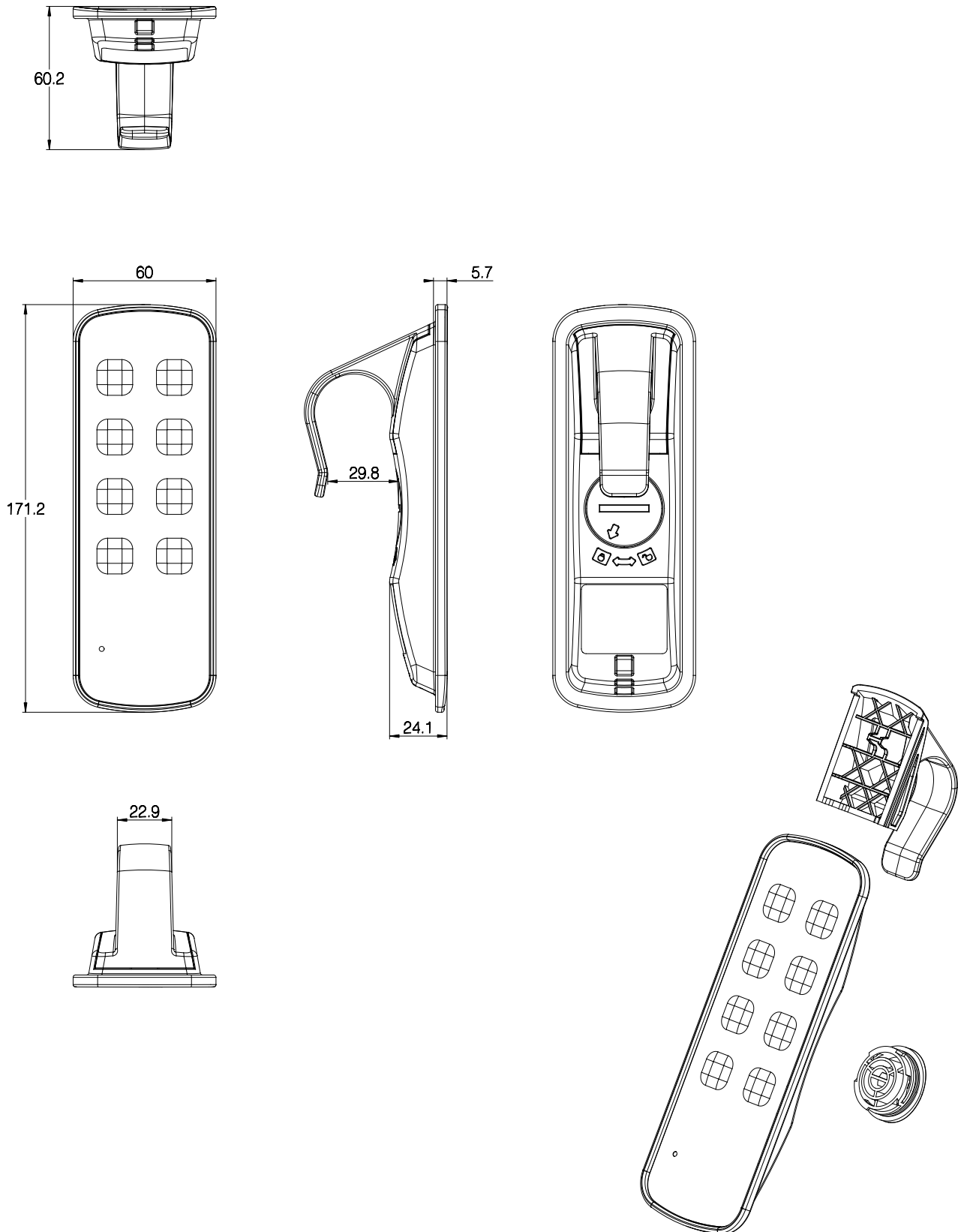


Drawing no.: 1031W4000

Functionality

Locking

Locking is possible with/without the magnet and pressing a key on the HB200. Please note that the way to lock must be defined in the control box software.

As an example to lock with magnet: hold the magnet key over the marking (©) and press an odd key number (Typically the up arrow). To unlock a row use the magnet key and press an even number (Typically the down arrow).

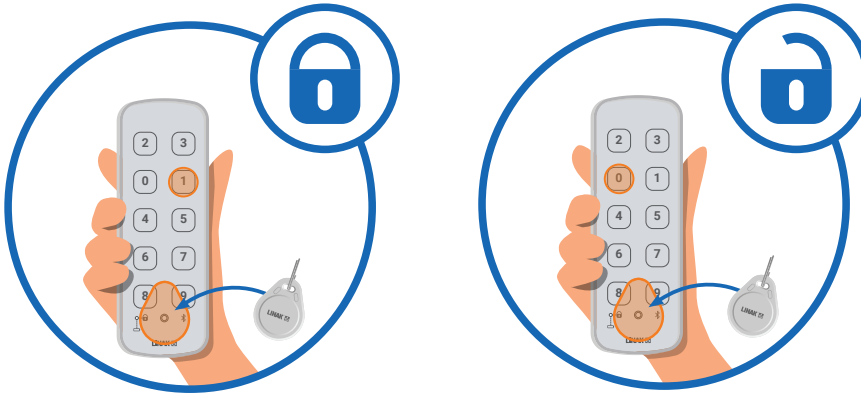

Magnet key

Remember to order magnet key:

Magnet key - ordering no. 0858008 (RAL 7035 light grey).



Battery


The battery in the HB200 is a standard CR2032 coin cell battery.

Battery lifetime

With a usage of 140 sec/day, the HB200 will last approximately two years.

Changing the battery

To change the HB200 battery, open the battery cover on the back using a coin or a similar tool to turn the arrow counterclockwise from the locked icon to the unlocked state.

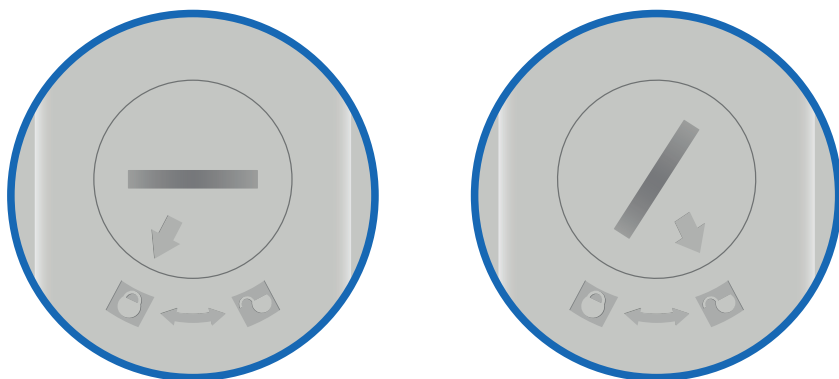

Low battery indication

When the battery power level falls below 20%, the LED will flash 4 times when a key is pressed.

New battery indication

When the battery has been changed, the LED will be lit for 4 seconds after the first keypress.



Battery cover

It is possible to order extra battery covers.

Battery cover ordering no. SA1031W9012




Pairing Bluetooth devices

Direct pairing

Direct pairing is used for pairing a LINAK control directly to a LINAK control box that supports BLE.


1. Enter pairing mode
2. Move the hand control closer to the control box you want to pair with
3. Pair the hand control with the control box

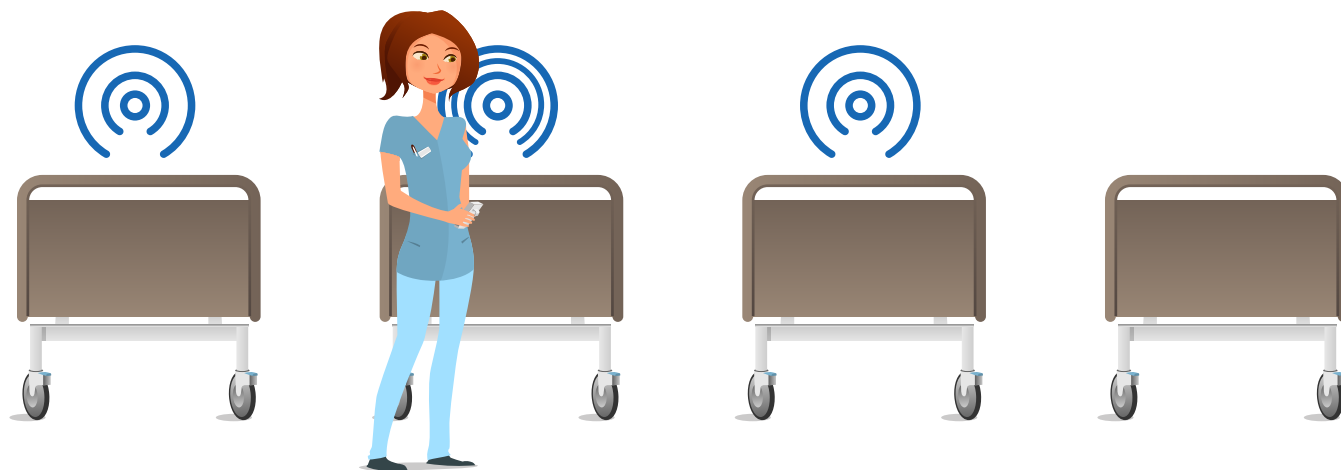

In pairing mode, the light/sound frequency will increase when the HB200 gets closer to a control box.

Connecting to the control box

When the hand control LED is blinking fast and the control box gives a high frequency sound in the same speed, the devices are ready for pairing. To finalise the pairing, press Key 1 on the hand control.

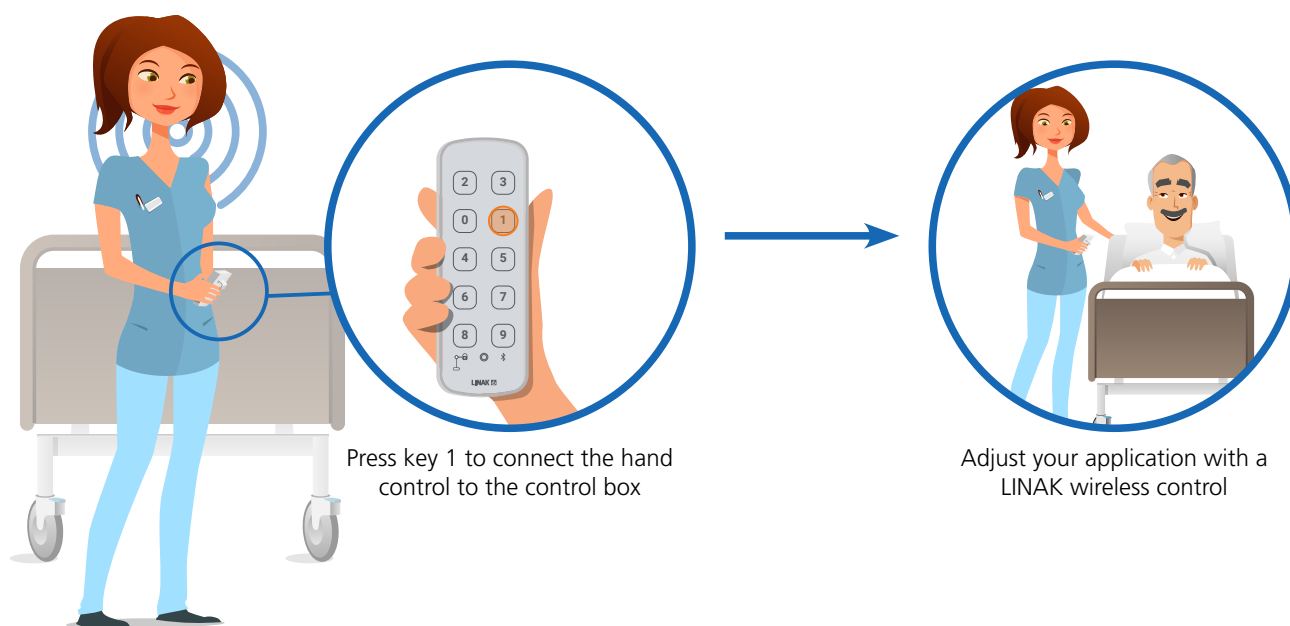


Press key 1 to connect the hand control to the control box

Adjust your application with a LINAK wireless control

Front covers


The HB200 is available in nine different standard front covers covering 1 to 5 row hand controls. Please contact LINAK A/S for customised front covers.

1 row

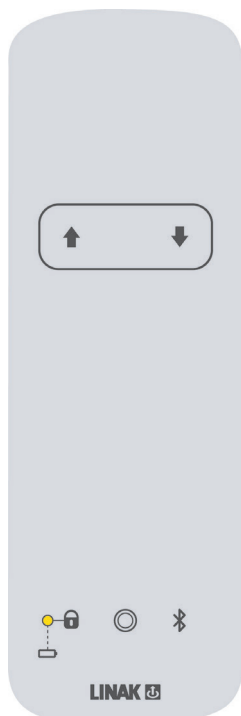


00HB2002A-002

2 rows

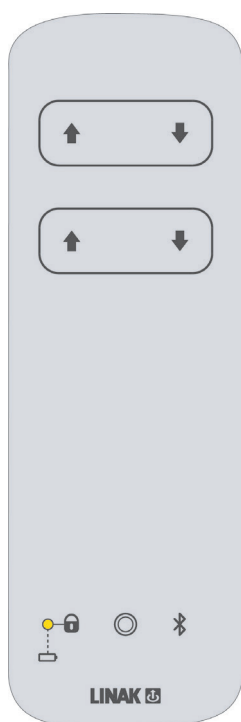


00HB2002B-002

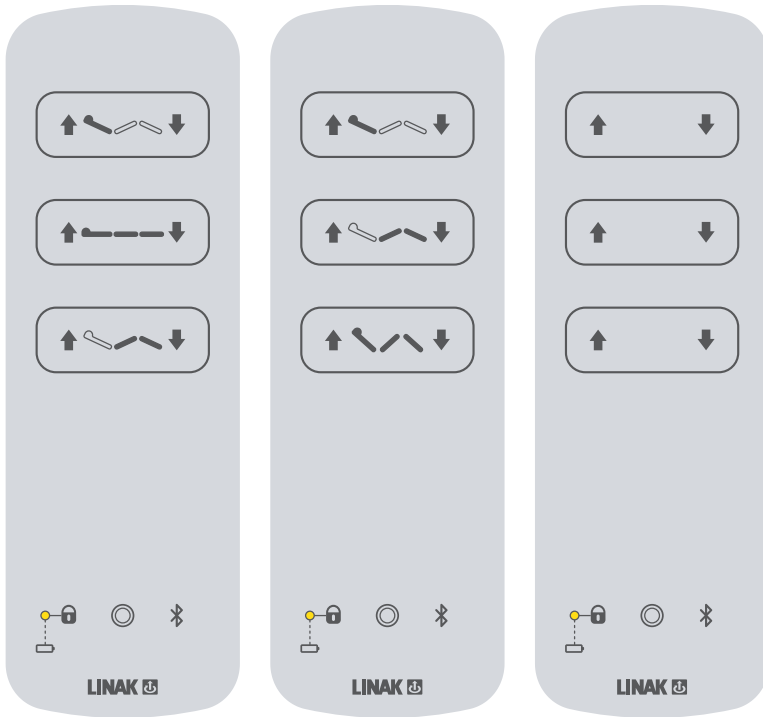3 rows

00HB2003B-001

00HB2003B-002


00HB2003A-003

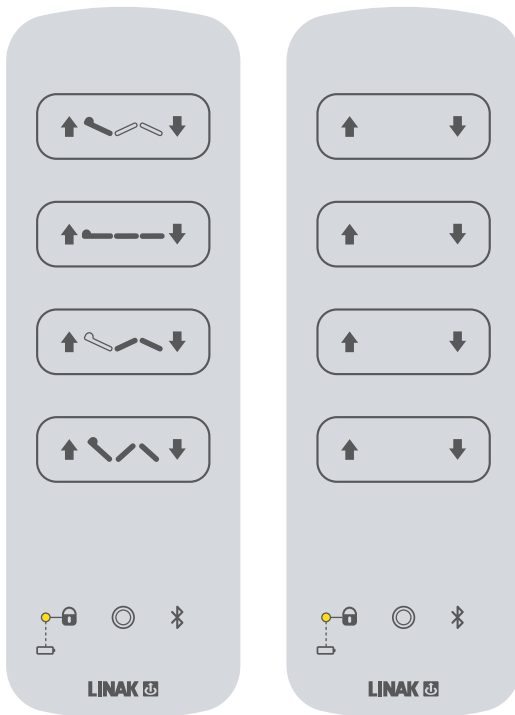4 rows


00HB2004B-001

00HB2004B-002


5 rows

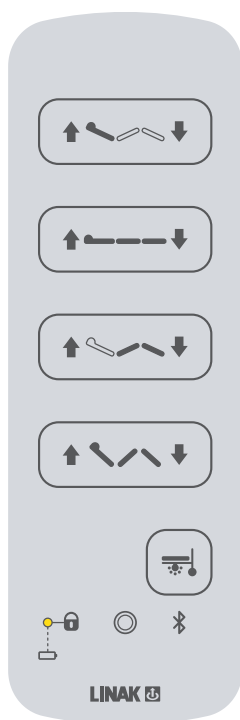

00HB2005B-002

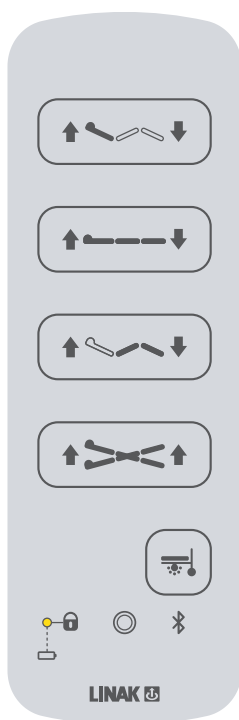

00HB2005B-003

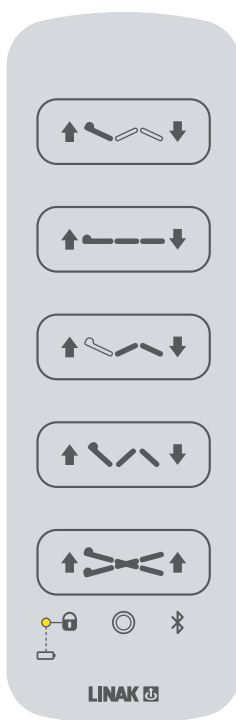

00HB2005B-001

Contacts

FACTORIES

Denmark - Headquarters

LINAK A/S

Phone: +45 73 15 15 15

Fax: +45 74 45 80 48

Fax (Sales): +45 73 15 16 13

Web: www.linak.com

China

LINAK (Shenzhen) Actuator Systems, Ltd.

Phone: +86 755 8610 6656

Phone: +86 755 8610 6990

Web: www.linak.cn

Slovakia

LINAK Slovakia s.r.o.

Phone: +421 51 7563 444

Web: www.linak.sk

Thailand

LINAK APAC Ltd.

Phone: +66 33 265 400

Web: www.linak.com

USA

LINAK U.S. Inc.

Americas Headquarters

Phone: +1 502 253 5595

Fax: +1 502 253 5596

Web: www.linak-us.com
www.linak-latinamerica.com

SUBSIDIARIES

Australia

LINAK Australia Pty. Ltd

Phone: +61 3 8796 9777

Fax: +61 3 8796 9778

E-mail: sales@linak.com.au

Web: www.linak.com.au

Austria

LINAK Zweigniederlassung - Österreich (Wien)

Phone: +43 (1) 890 7446

Fax: +43 (1) 890 744615

E-mail: info@linak.de

Web: www.linak.at - www.linak.hu

Belgium

LINAK Actuator-Systems NV/SA

(Belgium & Luxembourg)

Phone: +32 (0)9 230 01 09

E-mail: beinfo@linak.be

Web: www.linak.be - www.fr.linak.be

Brazil

LINAK Do Brasil Comércio De Atuadores Ltda.

Phone: +55 (11) 2832 7070

Fax: +55 (11) 2832 7060

E-mail: info@linak.com.br

Web: www.linak.com.br

Canada

LINAK Canada Inc.

Phone: +1 502 253 5595

Fax: +1 416 255 7720

E-mail: info@linak.ca

Web: www.linak-us.com

Czech Republic

LINAK C&S s.r.o.

Phone: +42 058 174 1814

Fax: +42 058 170 2452

E-mail: info@linak.cz

Web: www.linak.cz - www.linak.sk

Denmark - International

LINAK International

Phone: +45 73 15 15 15

E-mail: info@linak.com

Web: www.linak.com

Denmark - Sales

LINAK Danmark A/S

Phone: +45 86 80 36 11

Fax: +45 86 82 90 51

E-mail: linak@linak-silkeborg.dk

Web: www.linak.dk

Finland

LINAK OY

Phone: +358 10 841 8700

E-mail: linak@linak.fi

Web: www.linak.fi

France

LINAK France E.U.R.L

Phone: +33 (0) 2 41 36 34 34

Fax: +33 (0) 2 41 36 35 00

E-mail: linak@linak.fr

Web: www.linak.fr

Germany

LINAK GmbH

Phone: +49 6043 9655 0

Fax: +49 6043 9655 60

E-mail: info@linak.de

Web: www.linak.de

India

LINAK A/S India Liaison Office

Phone: +91 120 4531797

Fax: +91 120 4786428

E-mail: info@linak.in

Web: www.linak.in

Ireland

LINAK UK Limited (Ireland)

Phone: +44 (0)121 544 2211

Fax: +44 (0)121 544 2552

+44 (0)796 855 1606 (UK Mobile)

+35 387 634 6554 (Rep.of Ireland Mobile)

E-mail: sales@linak.co.uk

Web: www.linak.co.uk

Italy

LINAK ITALIA S.r.l.

Phone: +39 02 48 46 33 66

Fax: +39 02 48 46 82 52

E-mail: info@linak.it

Web: www.linak.it

Japan

LINAK K.K.

Phone: 81-45-533-0802

Fax: 81-45-533-0803

E-mail: linak@linak.jp

Web: www.linak.jp

Malaysia

LINAK Actuators Sdn. Bhd.

Phone: +60 4 210 6500

Fax: +60 4 226 8901

E-mail: info@linak-asia.com

Web: www.linak.my

Netherlands

LINAK Actuator-Systems B.V.

Phone: +31 76 5 42 44 40 /

+31 76 200 11 10

E-mail: info@linak.nl

Web: www.linak.nl

New Zealand

LINAK New Zealand Ltd

Phone: +64 9580 2071

Fax: +64 9580 2072

E-mail: nzsales@linak.com.au

Web: www.linak.com.au

Norway

LINAK Norge AS

Phone: +47 32 82 90 90

E-mail: info@linak.no

Web: www.linak.no

Poland

LINAK Polska

LINAK Danmark A/S (Spółka Akcyjna)

Phone: +48 22 295 09 70 /

+48 22 295 09 71

E-mail: info@linak.pl

Web: www.linak.pl

Republic of Korea

LINAK Korea Ltd.

Phone: +82 2 6231 1515

Fax: +82 2 6231 1516

E-mail: info@linak.kr

Web: www.linak.kr

Slovakia

LINAK Slovakia S.R.O.

Phone: +421 51 7563 444

Web: www.linak.sk

Spain

LINAK Actuadores, S.L.u

Phone: +34 93 588 27 77

Fax: +34 93 588 27 85

E-mail: esma@linak.es

Web: www.linak.es

Sweden

LINAK Scandinavia AB

Phone: +46 8 732 20 00

Fax: +46 8 732 20 50

E-mail: info@linak.se

Web: www.linak.se

Switzerland

LINAK AG

Phone: +41 43 388 31 88

E-mail: info@linak.ch

Web: www.linak.ch - www.fr.linak.ch

www.it.linak.ch

Taiwan

LINAK (Shenzhen) Actuator systems Ltd.

Taiwan Representative office

Phone: +886 2 272 90068

Fax: +886 2 272 90096

E-mail: sales@linak.com.tw

Web: www.linak.com.tw

Turkey

LINAK İth. İhr. San. ve Tic. A.Ş.

Phone: +90 312 4726338

Fax: +90 312 4726635

E-mail: info@linak.com.tr

Web: www.linak.com.tr

United Kingdom

LINAK UK Limited

Phone: +44 (0)121 544 2211

Fax: +44 (0)121 544 2552

E-mail: sales@linak.co.uk

Web: www.linak.co.uk

DISTRIBUTORS

Argentina

Novotec Argentina SRL

Phone: 011-4303-8989 / 8900

Fax: 011-4032-0184

E-mail: info@novotecargentina.com

Web: www.novotecargentina.com

Colombia

MEM Ltda

Phone: +[57] (1) 334-7666

Fax: +[57] (1) 282-1684

E-mail: servicioalcliente@memltda.com.co

Web: www.mem.net.co

India

Mechatronics Control Equipments India Pvt Ltd

Phone: +91-44-28558484, 85

E-mail: bala@mechatronicscontrol.com

Web: www.mechatronicscontrol.com

Indonesia

PT. Himalaya Everest Jaya

Phone: +6 221 544 8956 /+6 221 544 8965

Fax: +6 221 619 1925

Fax (Sales): +6 221 619 4658

E-mail: hejplastic-div@centrin.net.id

Web: www.hej.co.id

Israel

NetivTech LTD

Phone: +972 55-2266-535

Fax: +972 2-9900-560

Email: info@NetivTech.com

Web: www.netivtech.com

Singapore

Servo Dynamics Pte Ltd

Phone: +65 6844 0288

Fax: +65 6844 0070

E-mail: servodynamics@servo.com.sg

South Africa

Industrial Specialised Applications CC

Phone: +27 011 466 0346

E-mail: gartht@isagroup.co.za

Web: www.isaza.co.za

United Arab Emirates

Mechatronics

Phone: +971 4 267 4311

Fax: +971 4 267 4312

E-mail: mechtron@emirates.net.ae

Terms of use

LINAK® takes great care in providing accurate and up-to-date information on its products. However, the user is responsible for determining the suitability of LINAK products for a specific application. Due to continual development, LINAK products are subject to frequent modifications and changes. LINAK reserves the rights to conduct modifications, updates, and changes without any prior notice. For the same reason, LINAK cannot guarantee the correctness and actual status of imprinted information on its products.

LINAK uses its best efforts to fulfil orders. However, for the reasons mentioned above, LINAK cannot guarantee availability of any particular product at any given time. LINAK reserves the right to discontinue the sale of any product displayed on its website or listed in its catalogues or in other written material created and produced by LINAK, LINAK subsidiaries, or LINAK affiliates. All sales are subject to the 'Standard Terms of Sale and Delivery for LINAK A/S' available on LINAK websites. LINAK and the LINAK logotype are registered trademarks of LINAK A/S. All rights reserved.